

CleanCut™

Autopsy saws
and extraction
system



deSouther
MEDICAL



Comercial Hospitalaria Grupo-3, S.L.

Rúa Arroncal 9, Vial C, Nave 4C - Parque Empresarial Porto do Molle.
36350 Nigrán (Pontevedra) - Tlf. +34 986 493 253 - Fax. 986 425 165
chgrupo3@chgrupo3.com - www.chgrupo3.com

CleanCut™ saw systems

low voltage and air

The DeSoutter Medical CleanCut™ autopsy saw system incorporates a saw handpiece and an extractor system which operates to the highest standards of pathogenic debris containment and filtration.

This safe 42Vd.c. handpiece is fully immersible allowing it to be routinely scrubbed and disinfected after use.



The clear-view plastic extractor hood can be rotated to the most convenient working position. The 3 metre long extractor hose is smooth bored for efficient flushing and disinfecting.



Improved cutting access is allowed by the slim, extended nose incorporated on the handpiece.

A rotary knob, located on the control box, allows the saw oscillating speed to be varied from zero to maximum for optimum cutting efficiency. The control box is splash proof for easy cleaning.



Saw and control box system complete NS3A

NS3 saw specification

Oscillating speed range	0-14000 osc/min.
Voltage	42Vd.c.
Motor protection	Overload cut-out, auto reset
Mode of operation	Continuous
Cable length	4 metres
Length	275mm
Weight	1.15kg
Sound level	70dB(A)

Degree of protection against ingress of liquid
IP67-immersible at 1 metre



NS3 Electric Saw

NS170-T air driven oscillating saw

This saw has been specifically developed to allow repeated sterilization by vacuum steam autoclaving



Oscillating speed	17000 osc/min.
Air consumption	3.3l/s (7cfm).
Normal operating pressure	7bar (101lb/in ²)
Length	267mm
Weight	0.86kg
Sound level	75dB(A)
Order code	222664

See page 4 for saw blade and co-axial hose options.



The suction rate can be switched between powerful 'high' and quieter 'low' setting.

Extraction is switched on and off by the handpiece button. Once switched off, the extractor will run for a further 5 seconds to purge system of debris.

NS3 Saw and extractor system complete CNS3A

The extractor has five levels of filtration enabling the system to meet the highest health standards.

The first level is a disposable inner bag which retains debris when removed from the extractor for disposal.

Level 2 is a zipped cloth outer bag which retains debris if the inner bag fails. Alternatively it may be disposed of with inner bag without opening the zip.

Levels 3 and 4 comprise two cloth filters which protect the microfilter in the event of a splitor poorly fitted paper or cloth bag. They may be rinsed and soaked in disinfectant if required.

Level 5 is a microfilter which is located within the extractor unit's powerhead. It contains 99.997% of particles greater than 0.5 microns.

Control box specification

Supply voltages	120V~ or 230V~
Output to saw	42Vd.c.
Supply frequency	50-60Hz
S.R.S.	Safety reset system
Splashproof	IP64
Weight	2.95kg
Dimensions	240Lx128Hx120W

Extractor specification

Supply voltages	120V~ or 230V~
Motor power (max.)	1kW
Supply frequency	50-60Hz
Suction	2500 water gauge mm
Sound level	58dB(A)
Filtration	99.997% > 0.5 microns
Debris bag capacity	7.5 litres

NS3 saw and extractor system comprises:

System components		U.K. 230V UK plug	Euro 230V Euro plug	U.S. 120V U.S. plug
NS3 saw	Order code	1250444	1250444	1250444
Hose kit	Order code	5550	5550	5550
Extractor and control box	Order code	5280	7850	5270
Complete system	Order code	CNS3A-230UK	CNS3A-230EU	CNS3A-120

NS3 saw and control box with handles - less extraction - comprises:

NS3 saw	Order code	1250444	1250444	1250444
Control box	Order code	605903	613573	606763
Complete system	Order code	NS3A-230	NS3A-230EU	NS3A-120

CleanCut™ accessories







Saw blades and hoses

Saw blades

Sawblades shown actual size.

Steel, hard chrome plated.

*Blades supplied with saw.

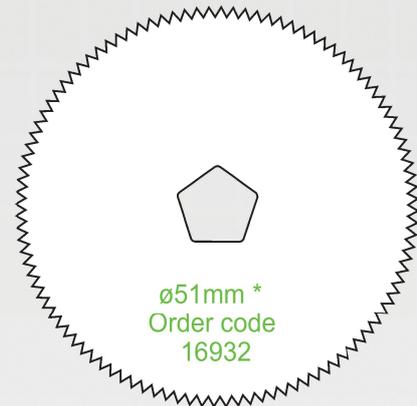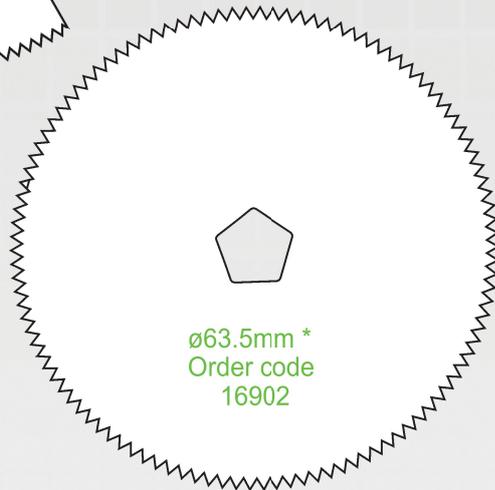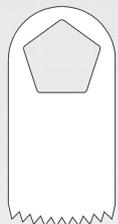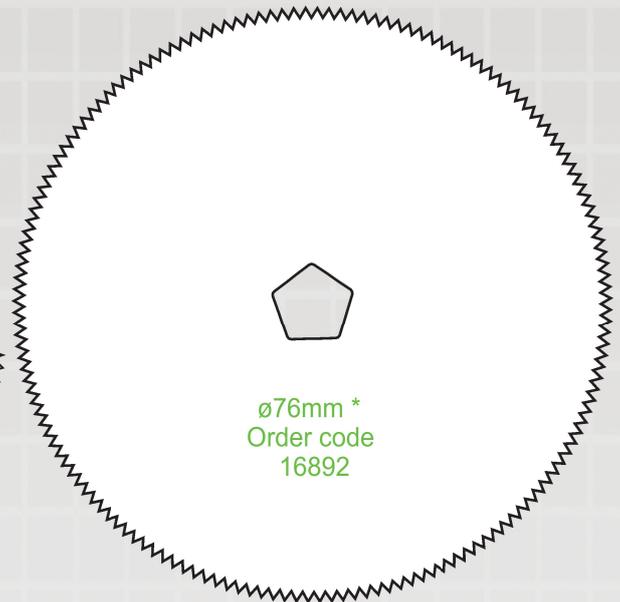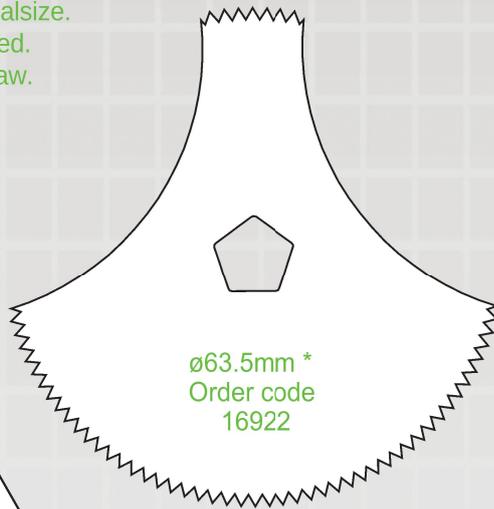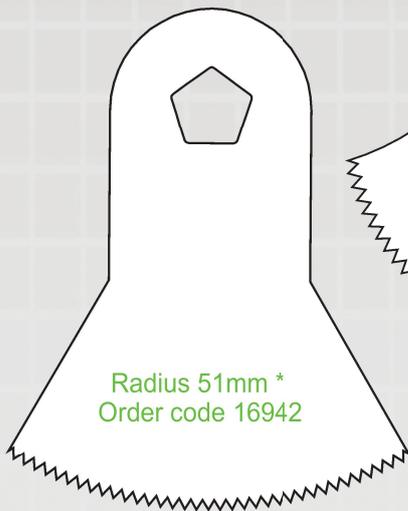

Accessories

3 metre NS3 saw extension cable - Order code 5500



Synthetic
inner debris bag -
Pack of 10
Order code 602763



Autopsy
outer debris bag
Order code 5190

Air supply hoses for NS170-T

Combined air supply and
exhaust co-axial hose
which features a special
quick action connector.

Supply end fitting - MA7.
Other probe types available.

3m length Order code 13000
4m length Order code 13100
5m length Order code 13200



116-EN-9



Comercial Hospitalaria Grupo-3, S.L.

Rúa Arroncal 9, Vial C, Nave 4C - Parque Empresarial Porto do Molle.
36350 Nigrán (Pontevedra) - Tlf. +34 986 493 253 - Fax. 986 425 165
chgrupo3@chgrupo3.com - www.chgrupo3.com

deSoutter
MEDICAL